


How to Access and Use the Parasight Portal


Step 1. Verify Parasight App software version

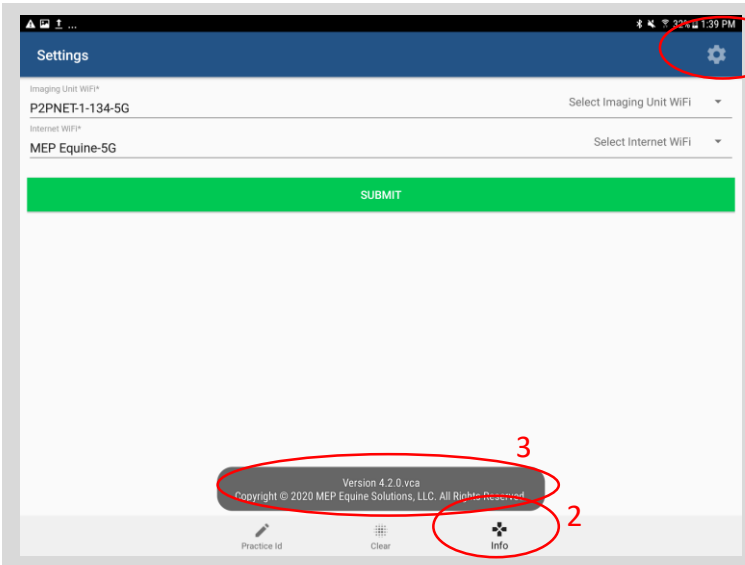

Version 4.2 or later is needed to use the Parasight Portal. Contact customer support for assistance if you need help updating/installing.


To determine which version you have installed on your tablet: Power on Parasight tablet, open Parasight App, tap the settings icon (1) at top right. Tap info icon at bottom right (2) to see version (3).

Step 2. Parasight Portal Access

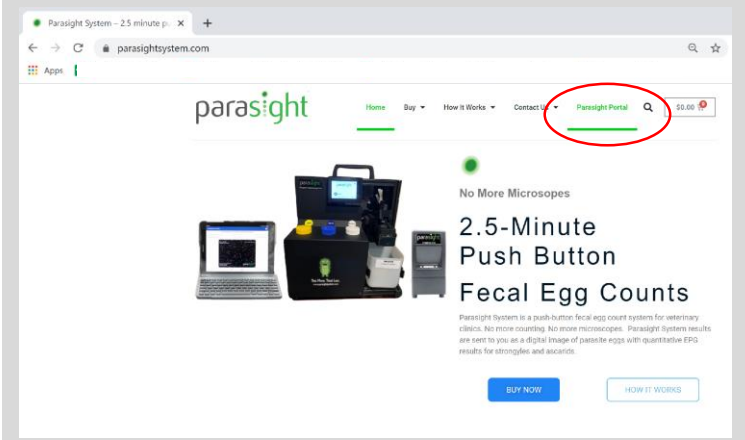

You can access the portal from any web Browser: tablet, mobile phone, or desktop computer.

From our website home page. Click on Parasight Portal.

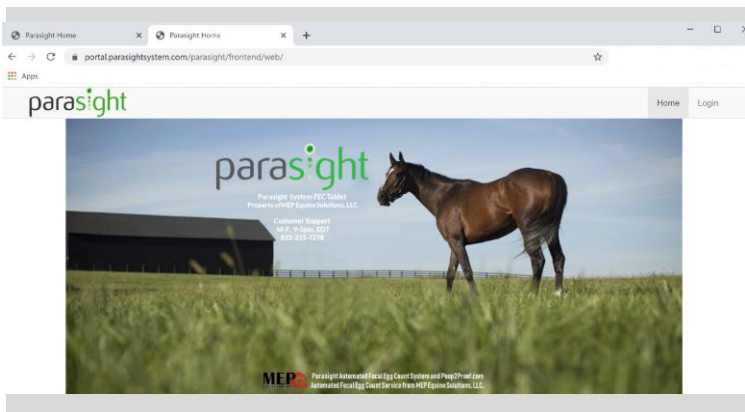

Or type into your browser:

<https://portal.parasightsystem.com>



How to Access and Use the Parasight Portal

Step 3. Login in to Parasight Portal

Login

Please fill out the following fields to login:

Username


Password

Remember Me

Your login information was emailed to the email of record. If you do not have your username and/or password please contact:
 support@mepequinesolutions.com or
 833-233-7278.

Internet access is needed to access the Parasight Portal.

Step 4. View, Email, Export, Print Results On Demand

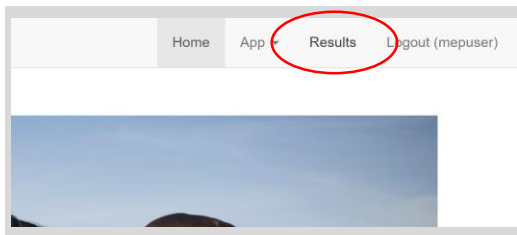

Results captured using version 4.2 or later will be accessible in the cloud. Click on Results to access. Click on actions icons to view or email results. Click on Export to download view or selected results as a CSV or Excel file.

Test Date	Vet First Name	Vet Last Name	Vet Title	Vet Company	Vet Zipcode	Client First Name	Client Last Name	Animal Name	Test Type	Strongyle Count	Ascarid Count	Treatment	Action
Feb 14, 2020 2:54:09	Dr	Vet	DVM	Vetcoo	90000	A	Client	Ed	FECRT	940	0	No Treatment	
Feb 13, 2020 6:17:21	Dr	Vet	DVM	Vetcoo	90000	A	Client	Ed	FEC	0	0	Consult With Your Vet	
Feb 13, 2020 5:46:17	Dr	Vet	DVM	Vetcoo	90000	A	Client	Ed	FEC	0	0	No Treatment	
Feb 10, 2020 4:31:29	Dr	Vet	DVM	Vetcoo	9000	A	Client	Ed	FEC	120	0	custom treatment	
Feb 04, 2020 23:11:52	Dr	Vet	DVM	Vet	9000	A	Client	Ed	FEC	180	0	Consult With Your Vet	

Action



View
 Email

Please view our video tutorial for tips and tricks on using Parasight Portal.

